

UVDI-360 Room Sanitizer

UV Disinfection Protocol | Operating Room



The UVDI-360 Room Sanitizer, as a supplement to standard terminal cleaning protocols, provides enhanced environmental disinfection. Please follow your facility's manual cleaning and disinfection procedures prior to use.

UV Disinfection Protocols:

Prior to getting started, don appropriate personal protective equipment, including gloves. Review the entire room to ensure there is no visible soil or dust present.



1. Completely remove mattress linens and straps if applicable.
Position the UVDI-360 Room Sanitizer to one side of the Operating table and ensure lamps, monitors, and other high-touch surfaces are facing it.




2. Ensure biohazardous waste bin, linen hamper and trash can face the UVDI-360 Room Sanitizer.




3. Program treatment cycle, exit and close the room door.



4. Place protective case shells outside each entry with Caution information clearly visible.

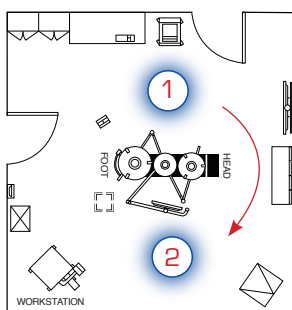

5. Move the UVDI-360 Room Sanitizer to the second position on the opposite side of the operating room and start the next cycle.

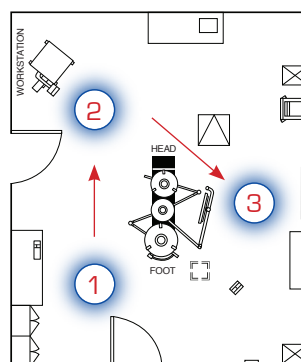

6. If room protocols require three placements, move the UVDI-360 Room Sanitizer to the next position and start a new cycle.

Once all cycles have been completed, remove the UVDI-360 Room Sanitizer and follow your room's protocols to complete the room preparation.